TECNIS-Torus-Rechner

Bedienungshandbuch

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Registration

Europe - Middle East - Africa English

I have read and agree to the terms below.

Accept

TECNIS® intraocular lenses are currently available for sale in select markets. This application is therefore intended only for residents of those markets in which the respective TECNIS® intraocular lens(es) are available and this application may only be used for those TECNIS® intraocular lens(es) that are approved for use in your market. Using this mobile application and clicking the “Accept” button constitutes your acknowledgement of these restrictions, and your acknowledgement that you selected your proper location of residence.

TECNIS® TORIC IOL CALCULATOR LICENSE AGREEMENT, TERMS OF USE AND PRIVACY POLICY

PLEASE READ THIS USE AND LICENSE AGREEMENT (“AGREEMENT”) IN FULL

[PRIVACY POLICY](#) [TERMS OF SERVICE](#) [CONTACT](#) [LRI CALCULATOR](#)

The product-specific application is intended for the residents of a particular country or countries. As a result, the application may contain information on pharmaceuticals, medical devices and other products or uses of those products that are not approved in other countries or regions. If you are a resident of a country other than those to which the application lists, please return to surgical.jnjvision.com or contact your local Johnson & Johnson Vision affiliate to obtain the appropriate product information for your country of residence.

This application is intended for use in connection with Johnson & Johnson Vision IOLs only and should not be used in connection with any other IOLs.

©2021 Johnson & Johnson Surgical Vision, Inc.

Unless otherwise specified, all product names appearing in this program are trademarks owned by or licensed to Johnson & Johnson Vision, its subsidiaries or affiliates. No use of any Johnson & Johnson Vision trademark, trade name, or trade dress in this program may be made without the prior written authorization of Johnson & Johnson Vision, except to identify the product or service of the company.

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The screenshot displays the 'TECNIS Family of IOLs' calculator interface. It is divided into several sections:

- Surgeon and Patient Information:** Fields for Surgeon Name, Patient Information, Eye Selection (OD/OS), and K Notation.
- Keratometry:** Fields for Surgically Induced Astigmatism (SIA), Flat K1, Steep K2, and Preop Corneal Astigmatism.
- Biometry:** Fields for Axial Length, Method (Optical or Immersion), and A-constant.
- Lens Selection List:** A central list of lens options categorized by type:
 - MONOFOCAL:** TECNIS Eyhance™ Toric II DIU, TECNIS Monofocal Toric II ZCU, TECNIS Monofocal Toric ZCT.
 - PRESBYOPIA CORRECTING:** TECNIS Synergy™ Toric II DFW, TECNIS Symphony™ Toric II OptiBlue™ ZXW, TECNIS Symphony™ Toric ZXT.
 - Multifocal:** TECNIS Multifocal Toric II ZKU (+2.75 D), TECNIS Multifocal Toric II ZLU (+3.25 D).
- Visual Representation:** A diagram of an eye with a circular axis scale (0° to 315°) and the text 'N a s a i'.
- Results Table:** A table for 'IOL Results' with columns for IOL Model, Orientation, Residual Astigmatism (Cylinder, Axis).

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TECNIS Eyhance™ Toric II DIU
with TECNIS SIMPLICITY™ Delivery System

TORIC CALCULATOR
United States
CHANGE LENS

Calculator Preferences & Support

Surgeon and Patient Information Notes...

Surgeon Name Date 3/24/2022

Patient Information Patient Age

Eye Selection OD (Right) OS (Left) K Notation D mm

Keratometry

Surgically Induced Astigmatism (SIA) D @ Meridian (Incision Location)

Flat K1 D Flat K1 @ Meridian

Steep K2 D Steep K2 @ Meridian

Preop Corneal Astigmatism D Include Posterior Corneal Astigmatism

Biometry

Axial Length mm

Method

A-constant 119.30

Calculation Preferences

SE IOL Power D

K Index

Refractive Cylinder Convention Plus Minus

Final Results

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis

Calculate Results Clear Entries Print Results

PRIVACY POLICY TERMS OF SERVICE CONTACT LRI CALCULATOR

¹ Koch DD et al. Contribution of posterior corneal astigmatism to total corneal astigmatism. J Cataract Refract Surg. Dez. 2012;38(12):2080-7

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Toric II
DIU

TORIC CALCULATOR
United States
CHANGE LENS

Calculator
Preferences & Support

Surgeon and Patient Information Notes...

Surgeon Name: Dr. Surgeon Date: 3/24/2022

Patient Information: XYZ-123-ABC Patient Age: 65

Eye Selection: OD (Right) OS (Left) K Notation: D mm

Keratometry

Surgically Induced Astigmatism (SIA): 0.35 D @ Meridian (Incision Location): 180

Flat K1: 41.25 D Flat K1 @ Meridian: 12

Steep K2: 43.34 D Steep K2 @ Meridian: 102

Preop Corneal Astigmatism: 2.09 D Include Posterior Corneal Astigmatism

Biometry

Axial Length: 24.11 mm

Method: Optical or Immersion

A-constant: 119.30

Calculation Preferences

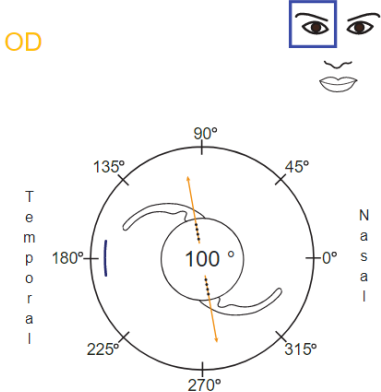
SE IOL Power: 20.0 D

K Index: 1.3375

Refractive Cylinder Convention: Plus Minus

Calculate Results
Clear Entries
Print Results

OD

Final Results

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
<input type="radio"/> DIU300	100 °	+0.38 D	100 °
<input type="radio"/> DIU375	100 °	+0.11 D	10 °
<input type="radio"/> DIU450	100 °	+0.61 D	10 °

PRIVACY POLICY
TERMS OF SERVICE
CONTACT
LRI CALCULATOR

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Toric II
DIU


Return to Calculator

Surgeon Name	Patient Information	Date	Patient Age
Dr. Surgeon	XYZ-123-ABC	3/24/2022	65

OD (Right)

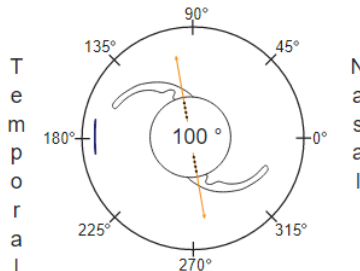T
e
m
p
o
r
a
l

N
a
s
a
l

— (Incision Location) 180°
— Calculated Orientation

IOL Details		Residual Refraction	
IOL Model	Orientation	Cylinder	Axis
DIU375	100 °	+0.11 D	10 °

Calculation Details		
SE IOL Power	K Index	Refractive Cylinder Convention
+20.0	1.3375	Plus

Keratometry and Biometry						
Flat K1 @ Meridian	Steep K2	SIA @ Meridian	AxL Length	Biometry Method	A-constant	PCA
41.25@12°	43.34@102°	0.35@180°	24.11	Optical or Immersion	119.30	No

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
DIU300	100 °	+0.38 D	100 °
DIU375	100 °	+0.11 D	10 °
DIU450	100 °	+0.61 D	10 °

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The screenshot displays the LRI Calculator interface with a 'Clear Entries' dialog box overlaid. The dialog box contains the following text:

Clear Entries

Please confirm that you wish to clear all entries that you have made on this screen. (e.g., OD (Right) & OS (Left))

OK Cancel

The background interface includes the following sections:

- Surgeon and Patient Information:** Surgeon Name (Dr. Surgeon), Date (3/24/2022), Patient Information (XYZ-123-ABC), Patient Age (85), Eye Selection (OD (Right)), K Notation (D).
- Keratometry:** Surgically Induced Astigmatism (SIA) (0.35 D), Flat K1 (41.25 D), Steep K2 (43.34 D), Preop Corneal Astigmatism (2.09 D).
- Biometry:** Axial Length (24.11 mm), Method (Optical or Immersion), A-constant (119.30).
- Calculation Preferences:** SE IOL Power (20.0 D), K Index (1.3375), Refractive Cylinder Convention (Plus).
- Final Results:** IOL Details (IOL Model, Orientation) and Residual Astigmatism (Cylinder, Axis).

At the bottom of the interface, there are buttons for 'Calculate Results', 'Clear Entries', and 'Print Results', along with links for 'PRIVACY POLICY', 'TERMS OF SERVICE', 'CONTACT', and 'LRI CALCULATOR'.

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The screenshot shows the 'Preferences & Support' page of the TECNIS TORIC CALCULATOR. The page is divided into two main sections: 'Preferences' and 'Support'. The 'Preferences' section is further divided into 'Display', 'Calculation', and 'Method' sub-sections. The 'Display' section includes options for Location (United States), Language (English), K Notation (D), and Decimal (0.00). The 'Calculation' section includes options for Refractive Cylinder Convention (Plus) and K Index (-Select-). The 'Method' section includes an option for Optical or Immersion. The 'Support' section includes links for Training and Information Documents (TECNIS® Toric IOL Calculator User Tutorial, TECNIS® Toric IOL Calculator FAQ, TECNIS® Toric IOL Calculator Terms of Service). The page also features a footer with links for Privacy Policy, Terms of Service, Contact, and LRI CALCULATOR. The page number (01)05050474708822(8012) - 4.5.2.0 CE is also visible.

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The screenshot displays the 'Preferences & Support' page of the TECNIS TORIC CALCULATOR. The interface is divided into two main sections: 'Preferences' and 'Support'. The 'Preferences' section includes settings for Display (Location: United States, Language: English), K Notation (D, mm), Decimal (0.00, 0,00), Calculation (Refractive Cylinder Convention: Plus, Minus), Method (Optical or Immersion), and A-constant (119.30). The 'Support' section lists training and information documents: TECNIS® Toric IOL Calculator User Tutorial, TECNIS® Toric IOL Calculator FAQ, and TECNIS® Toric IOL Calculator Terms of Service. The version number (01)05050474708822(8012) - 4.5.2.0 is displayed at the bottom right of the Support section. The footer contains links for PRIVACY POLICY, TERMS OF SERVICE, CONTACT, and LRI CALCULATOR.

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Johnson & Johnson Surgical Vision, Inc.
31 Technology Drive, Suite 200,
Irvine, CA 92618 USA

AMO Ireland
Block B
Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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AMO Australia Pty Ltd.
1-5 Khartoum Road,
North Ryde, NSW 2113
Australia

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© 2023 Johnson & Johnson Surgical Vision, Inc.

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AMO Switzerland GmbH