TECNIS™-Torus-Rechner

Benutzerlernprogramm

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Registration

Europe - Middle East - Africa English

I have read and agree to the terms below.

Accept

TECNIS™ intraocular lenses are currently available for sale in select markets. This application is therefore intended only for residents of those markets in which the respective TECNIS™ intraocular lens(es) are available, and this application may only be used for those TECNIS™ intraocular lens(es) that are approved for use in your market. Using this mobile application and clicking the “Accept” button constitutes your acknowledgement of these restrictions, and your acknowledgement that you selected your proper location of residence.

TECNIS™ TORIC CALCULATOR LICENSE AGREEMENT, TERMS OF USE AND PRIVACY POLICY

PLEASE READ THIS USE AND LICENSE AGREEMENT (“AGREEMENT”) IN FULL

TERMS OF SERVICE LRI CALCULATOR

The product-specific application is intended for the residents of a particular country or countries. As a result, the application may contain information on pharmaceuticals, medical devices and other products or uses of those products that are not approved in other countries or regions. If you are a resident of a country other than those to which the application lists, please return to surgical.jnjvision.com or contact your local Johnson & Johnson Vision affiliate to obtain the appropriate product information for your country of residence.

This application is intended for use in connection with Johnson & Johnson Vision IOLs only and should not be used in connection with any other IOLs.

©2024 Johnson & Johnson Surgical Vision, Inc.

Unless otherwise specified, all product names appearing in this program are trademarks owned by or licensed to Johnson & Johnson Vision, its subsidiaries or affiliates. No use of any Johnson & Johnson Vision trademark, trade name, or trade dress in this program may be made without the prior written authorization of Johnson & Johnson Vision, except to identify the product or service of the company.

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The screenshot displays the 'TECNIS™ Toric Calculator' interface. At the top left, the 'TECNIS™ Family of IOLs' logo is highlighted with a blue box. The main content area is divided into several sections:

- Surgeon and Patient Information:** Fields for Surgeon Name, Patient Information, Eye Selection (OD/OS), and K Notation.
- Keratometry:** Fields for Surgically Induced Astigmatism (SIA), Flat K1, Steep K2, and Preop Corneal Astigmatism.
- Biometry:** Fields for Axial Length, Method (Optical or Immersion), and A-constant.
- SELECT LENS:** A dropdown menu showing a list of lens options:
 - TECNIS Eyhance™ with TECNIS SIMPLICITY™ (DIU)
 - TECNIS™ Toric II 1-Piece IOL (ZCU)
 - TECNIS™ Toric 1-Piece IOL (ZCT)
 - TECNIS PureSee™ with TECNIS SIMPLICITY™ (DET)
 - TECNIS Synergy™ with TECNIS SIMPLICITY™ (DFW)
 - TECNIS Symphony™ with TECNIS SIMPLICITY™ (DXW)
 - TECNIS Symphony™ (ZXT)
 - TECNIS™ Multifocal (ZMT)
- Visual Aids:** A diagram of an eye and a circular meridian chart with angles (90°, 45°, 0°, 315°, 270°, 225°, 180°, 135°).

At the bottom, there are buttons for 'Calculate Results', 'Clear Entries', and 'Print Results'. The footer contains 'TERMS OF SERVICE' and 'LRI CALCULATOR'.

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The screenshot displays the TECNIS™ Toric Calculator interface. The top navigation bar includes the TECNIS Eyhance™ logo, the text 'DIU', the region 'Europe - Middle East - Africa', and the Johnson & Johnson VISION logo. Below this is a 'CHANGE LENS' button. The main interface is divided into two tabs: 'Calculator' (active) and 'Preferences & Support'. The 'Calculator' tab contains several sections: 'Surgeon and Patient Information' with fields for name, date, patient information, and age; 'Keratometry' with fields for surgically induced astigmatism (SIA), flat K1, steep K2, and preop corneal astigmatism; 'Biometry' with fields for axial length, method, and A-constant; and 'Calculation Preferences' with dropdowns for SE IOL Power, K Index, and Refractive Cylinder Convention. To the right of the input fields is a diagram of an eye with meridians labeled from 0° to 315° in 45° increments. Below the diagram is a 'Final Results' table with columns for IOL Model, Orientation, Residual Astigmatism (Cylinder), and Axis. At the bottom of the calculator section are buttons for 'Calculate Results', 'Clear Entries', and 'Print Results'. The footer contains 'TERMS OF SERVICE' and 'LRI CALCULATOR'.

¹ Koch DD et al. Contribution of posterior corneal astigmatism to total corneal astigmatism. J Cataract Refract Surg. Dez. 2012;38(12):2080-7

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The screenshot displays the TECNIS™ Toric Calculator interface. The top navigation bar includes the TECNIS Eyhance™ logo, the model name 'DIU', and the Johnson & Johnson VISION logo. The interface is divided into 'Calculator' and 'Preferences & Support' sections.

Surgeon and Patient Information: Surgeon Name (Surgeon), Date (11/14/2024), Patient Information (XYZ-123-ABC), Patient Age (65). Eye Selection: OD (Right) selected, OS (Left) unselected. K Notation: D selected, mm unselected.

Keratometry: Surgically Induced Astigmatism (SIA) 0.30 D @ Meridian (Incision Location) 180°. Flat K1 41.25 D Flat K1 @ Meridian 12. Steep K2 43.34 D Steep K2 @ Meridian 102. Preop Corneal Astigmatism 2.09 D. Include Posterior Corneal Astigmatism: unchecked.

Biometry: Axial Length 24.11 mm. Method: Optical or Immersion. A-constant 119.30.

Calculation Preferences: SE IOL Power 20.0 D. K Index 1.3375. Refractive Cylinder Convention: Plus selected, Minus unselected.

Final Results Table:

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
<input type="radio"/> DIU300	101 °	+0.33 D	101 °
<input type="radio"/> DIU375	101 °	+0.16 D	11 °
<input type="radio"/> DIU450	101 °	+0.65 D	11 °

The interface also features a diagram of the eye showing the incision location (blue) and the postoperative astigmatism axis (orange/red) at 101°. The diagram is labeled with 'Temporal' and 'Nasal' directions. Buttons for 'Calculate Results', 'Clear Entries', and 'Print Results' are visible at the bottom.

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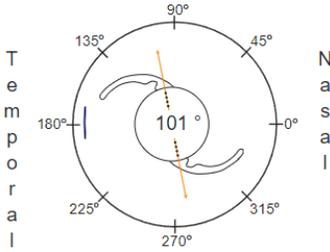
DIU

[Return to Calculator](#)

Surgeon Name	Patient Information	Date	Patient Age
Surgeon	XYZ-123-ABC	2/6/2024	65

OD (Right)

— (Incision Location) 180°
— Calculated Orientation

IOL Details		Residual Refraction	
IOL Model	Orientation	Cylinder	Axis
DIU300	101 °	+0.33 D	101 °

Calculation Details		
SE IOL Power	K Index	Refractive Cylinder Convention
+20.0	1.3375	Plus

Keratometry and Biometry						
Flat K1 @ Meridian	Steep K2	SIA @ Meridian	AxL Length	Biometry Method	A-constant	PCA
41.25@12°	43.34@102°	0.30@180°	24.11	Optical or Immersion	119.30	No

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
DIU300	101 °	+0.33 D	101 °
DIU375	101 °	+0.16 D	11 °
DIU450	101 °	+0.65 D	11 °

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The screenshot shows the TECNIS Eyhance LRI Calculator interface. A dialog box titled "Clear Entries" is open, asking for confirmation to clear all entries. The background interface includes sections for "Surgeon and Patient", "Keratometry", "Biometry", "Calculation Preferences", and "Final Results".

Surgeon and Patient

Surgeon Name: Surgeon Date: 11/14/2024
Patient Information: XYZ-123-ABC Patient Age: 65
Eye Selection: OD (Right) OS (Left) K Notation: D mm

Keratometry

Surgically Induced Astigmatism (SIA): 0.30 D @ Meridian (Incision Location): 180°
Flat K1: 41.25 D Flat K1 @ Meridian: 12°
Steep K2: 43.34 D Steep K2 @ Meridian: 102°
Preop Corneal Astigmatism: 2.09 D Include Posterior Corneal Astigmatism

Biometry

Axial Length: 24.11 mm
Method: Optical or Immersion
A-constant: 119.30

Calculation Preferences

SE IOL Power: 20.0 D
K Index: 1.3375
Refractive Cylinder Convention: Plus Minus

Final Results

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis

Buttons: Calculate Results, Clear Entries, Print Results

Footer: TERMS OF SERVICE, LRI CALCULATOR

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TECNIS™ Family of IOLs
TECNIS™ Toric Calculator
Europe - Middle East - Africa
Johnson & Johnson VISION

Calculator | Preferences & Support

Reset To Defaults | Apply Preferences

Preferences

Display

Location: Europe - Middle East - Africa | Language: English | K Notation: D mm | Decimal: 0.00 0,00

Calculation

Refractive Cylinder Convention: Plus Minus | K Index: -Select- | Method: Optical or Immersion | A-constant: 119.30

Include Posterior Corneal Astigmatism

Support

Training and Information Documents

- [TECNIS™ Toric Calculator User Tutorial](#)
- [TECNIS™ Toric Calculator Frequently Asked Questions \(FAQ\)](#)
- [TECNIS™ Toric Calculator Terms of Service](#)

Johnson & Johnson Surgical Vision, Inc.
31 Technology Drive, Suite 200
Irvine, CA 92618 USA
2024-11-14

AMO Ireland
Block B
Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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Support

Training and Information Documents

[TECNIS™ Toric Calculator User Tutorial](#)
[TECNIS™ Toric Calculator Frequently Asked Questions \(FAQ\)](#)
[TECNIS™ Toric Calculator Terms of Service](#)

Johnson & Johnson Surgical Vision, Inc.
31 Technology Drive, Suite 200
Irvine, CA 92618 USA
2024-11-14

AMO Ireland
Block B
Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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Johnson & Johnson Surgical Vision, Inc.
31 Technology Drive, Suite 200,
Irvine, CA 92618 USA

AMO Ireland
Block B

Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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AMO Australia Pty Ltd.
1-5 Khartoum Road,
North Ryde, NSW 2113
Australia

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© 2024 Johnson & Johnson Surgical Vision, Inc.

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AMO Switzerland GmbH