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The screenshot shows the 'Calculator' page for selecting IOLs. The interface includes the following elements:

- TECNIS Family of IOLs** logo at the top left.
- Calculator** header.
- Surgeon and Patient Information** section with fields for Surgeon Name, Patient Information, Eye Selection (OD/OS), and K Notation.
- Keratometry** section with fields for Surgically Induced Astigmatism (SIA), Flat K1, Steep K2, and Preop Corneal Astigmatism.
- Biometry** section with fields for Axial Length, Method (Optical or Immersion), and A-constant.
- MONOFOCAL** section listing options like Eyhance™ DIU, TECNIS Monofocal ZCU, and TECNIS Monofocal ZCT.
- PRESBYOPIA CORRECTING** section listing options like TECNIS Synergy™ DFW, TECNIS Symphony™ ZXW, and TECNIS Symphony™ ZXT.
- Results** table with columns for IOL Model, Orientation, Cylinder, and Axis.
- Johnson & Johnson VISION** logo and a diagram of the eye's meridians.
- Footer links for **PRIVACY POLICY**, **TERMS**, and **LRI CALCULATOR**.

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The screenshot shows the 'TORIC CALCULATOR' interface for TECNIS Eyhance Toric II DIU. The page is split into 'Calculator' and 'Preferences & Support' tabs. The 'Calculator' tab is active and contains several input sections:

- Surgeon and Patient Information:** Includes fields for Surgeon Name, Date (3/24/2022), Patient Information, Patient Age, Eye Selection (OD/OS), and K Notation (D/mm).
- Keratometry:** Includes fields for Surgically Induced Astigmatism (SIA), Flat K1, Steep K2, and Preop Corneal Astigmatism. There is a checkbox for 'Include Posterior Corneal Astigmatism'.
- Biometry:** Includes fields for Axial Length (mm), Method (Optical or Immersion), and A-constant (119.30).
- Calculation Preferences:** Includes dropdowns for SE IOL Power and K Index, and radio buttons for Refractive Cylinder Convention (Plus/Minus).

On the right side, there is a patient face icon with a blue box around the right eye, labeled 'OD'. Below it is a circular diagram for astigmatism orientation with angles from 0° to 315° in 45° increments. The 'Final Results' section shows a table with columns for IOL Model, Orientation, Residual Astigmatism (Cylinder, Axis).

¹ Koch DD et al. Contribution of posterior corneal astigmatism to total corneal astigmatism. J Cataract Refract Surg. 2012 Dec; 38(12):2080-7

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TECNIS Eyhance™ with TECNIS SIMPLICITY™ Delivery System
Toric II DIU
 TORIC CALCULATOR
 United States
 CHANGE LENS

Calculator Preferences & Support

Surgeon and Patient Information Notes...

Surgeon Name: Dr. Surgeon Date: 3/24/2022
 Patient Information: XYZ-123-ABC Patient Age: 65
 Eye Selection: OD (Right) OS (Left) K Notation: D mm

Keratometry

Surgically Induced Astigmatism (SIA): 0.35 D @ Meridian (Incision Location): 180°
 Flat K1: 41.25 D Flat K1 @ Meridian: 12°
 Steep K2: 43.34 D Steep K2 @ Meridian: 102°
 Preop Corneal Astigmatism: 2.09 D Include Posterior Corneal Astigmatism

Biometry

Axial Length: 24.11 mm
 Method: Optical or Immersion
 A-constant: 119.30

Calculation Preferences

SE IOL Power: 20.0 D
 K Index: 1.3375
 Refractive Cylinder Convention: Plus Minus

Calculate Results Clear Entries Print Results

Final Results

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
<input type="radio"/> DIU300	100°	+0.38 D	100°
<input type="radio"/> DIU375	100°	+0.11 D	10°
<input type="radio"/> DIU450	100°	+0.61 D	10°

PRIVACY POLICY TERMS OF SERVICE CONTACT LRI CALCULATOR

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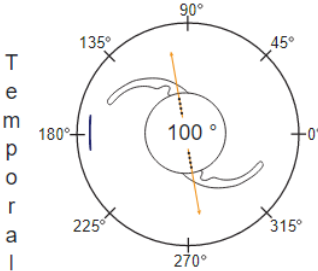
Return to Calculator

Surgeon Name	Patient Information	Date	Patient Age
Dr. Surgeon	XYZ-123-ABC	3/24/2022	65

OD (Right)

T e m p o r a l N a s a l

— (Incision Location) 180°
— Calculated Orientation

IOL Details		Residual Refraction	
IOL Model	Orientation	Cylinder	Axis
DIU375	100 °	+0.11 D	10 °

Calculation Details		
SE IOL Power	K Index	Refractive Cylinder Convention
+20.0	1.3375	Plus

Keratometry and Biometry						
Flat K1 @ Meridian	Steep K2	SIA @ Meridian	AxL Length	Biometry Method	A-constant	PCA
41.25@12°	43.34@102°	0.35@180°	24.11	Optical or Immersion	119.30	No

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
DIU300	100 °	+0.38 D	100 °
DIU375	100 °	+0.11 D	10 °
DIU450	100 °	+0.61 D	10 °

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The screenshot displays the LRI Calculator interface with a 'Clear Entries' dialog box overlaid. The dialog box contains the following text:

Clear Entries

Please confirm that you wish to clear all entries that you have made on this screen. (e.g., OD (Right) & OS (Left))

OK Cancel

The background interface includes the following sections:

- Surgeon and Patient Information:** Surgeon Name (Dr. Surgeon), Date (3/24/2022), Patient Information (XYZ-123-ABC), Patient Age (65), Eye Selection (OD (Right)), K Notation (D).
- Keratometry:** Surgically Induced Astigmatism (SIA) (0.35 D @ Meridian (Incision Location) 180), Flat K1 (41.25 D Flat K1 @ Meridian 12), Steep K2 (43.34 D Steep K2 @ Meridian 102), Preop Corneal Astigmatism (2.09 D). Includes a checkbox for 'Include Posterior Corneal Astigmatism'.
- Biometry:** Axial Length (24.11 mm), Method (Optical or Immersion), A-constant (119.30).
- Calculation Preferences:** SE IOL Power (20.0 D), K Index (1.3375), Refractive Cylinder Convention (Plus).
- Final Results:** IOL Details table with columns for IOL Model, Orientation, Residual Astigmatism (Cylinder, Axis).

At the bottom of the interface, there are buttons for 'Calculate Results', 'Clear Entries', and 'Print Results', along with links for 'PRIVACY POLICY', 'TERMS OF SERVICE', 'CONTACT', and 'LRI CALCULATOR'.

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Preferences Reset To Defaults Apply Preferences

Display

Location: United States Language: English K Notation: D mm
Decimal: 0.00 0,00

Calculation

Refractive Cylinder Convention: Plus Minus K Index: -Select-
Method: Optical or Immersion A-constant: 119.30

Include Posterior Corneal Astigmatism

Support

Training and Information Documents
[TECNIS® Toric IOL Calculator User Tutorial](#)
[TECNIS® Toric IOL Calculator FAQ](#)
[TECNIS® Toric IOL Calculator Terms of Service](#)

(01)05050474708822(8012) - 4.5.2.0 ©

PRIVACY POLICY TERMS OF SERVICE CONTACT LRI CALCULATOR

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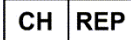
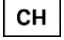
The screenshot displays the 'TECNIS TORIC CALCULATOR' interface for the 'United States'. The page is divided into two main sections: 'Preferences' and 'Support'. The 'Support' section is highlighted with a blue border and contains three links: 'TECNIS® Toric IOL Calculator User Tutorial', 'TECNIS® Toric IOL Calculator FAQ', and 'TECNIS® Toric IOL Calculator Terms of Service'. The 'Preferences' section includes settings for 'Display' (Location: United States, Language: English, K Notation: D, Decimal: 0.00), 'Calculation' (Refractive Cylinder Convention: Plus, K Index: -Select-), and 'Method' (Optical or Immersion). A footer contains 'PRIVACY POLICY', 'TERMS OF SERVICE', 'CONTACT', and 'LRI CALCULATOR'.

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Johnson & Johnson Surgical Vision, Inc.
31 Technology Drive, Suite 200,
Irvine, CA 92618 USA

AMO Ireland

Block B
Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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AMO Australia Pty Ltd.
1-5 Khartoum Road,
North Ryde, NSW 2113
Australia

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© 2023 Johnson & Johnson Surgical Vision, Inc.

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AMO Switzerland GmbH