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TECNIS[®] TORIC CALCULATOR
Family of IOLs *Johnson & Johnson* VISION

Registration

Europe - Middle East - Africa English I have read and agree to the terms below.

Accept

TECNIS[®] intraocular lenses are currently available for sale in select markets. This application is therefore intended only for residents of those markets in which the respective TECNIS[®] intraocular lens(es) are available and this application may only be used for those TECNIS[®] intraocular lens(es) that are approved for use in your market. Using this mobile application and clicking the "Accept" button constitutes your acknowledgement of these restrictions, and your acknowledgement that you selected your proper location of residence.

TECNIS[®] TORIC IOL CALCULATOR LICENSE AGREEMENT, TERMS OF USE AND PRIVACY POLICY

[PRIVACY POLICY](#) [TERMS OF SERVICE](#) [CONTACT](#) [LRI CALCULATOR](#)

The product-specific application is intended for the residents of a particular country or countries. As a result, the application may contain information on pharmaceuticals, medical devices and other products or uses of those products that are not approved in other countries or regions. If you are a resident of a country other than those to which the application lists, please return to surgical.jnjvision.com or contact your local Johnson & Johnson Vision affiliate to obtain the appropriate product information for your country of residence.

This application is intended for use in connection with Johnson & Johnson Vision IOLs only and should not be used in connection with any other IOLs.

©2021 Johnson & Johnson Surgical Vision, Inc.

Unless otherwise specified, all product names appearing in this program are trademarks owned by or licensed to Johnson & Johnson Vision, its subsidiaries or affiliates. No use of any Johnson & Johnson Vision trademark, trade name, or trade dress in this program may be made without the prior written authorization of Johnson & Johnson Vision, except to identify the product or service of the company.

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The screenshot displays the 'Lens Selection' interface of the TECNIS IOL calculator. The interface is organized into several functional areas:

- Surgeon and Patient Information:** Fields for Surgeon Name, Patient Information, Eye Selection (OD/OS), and K Notation.
- Keratometry:** Fields for Surgically Induced Astigmatism (SIA), Flat K1, Steep K2, and Preop Corneal Astigmatism.
- Biometry:** Fields for Axial Length, Method (Optical or Immersion), and A-constant.
- Lens Selection:** A central panel showing a list of lens options categorized by 'MONOFOCAL' and 'PRESBYOPIA CORRECTING'. The 'MONOFOCAL' section includes 'Eyhance™ DIU', 'Monofocal ZCU', and 'Monofocal ZCT'. The 'PRESBYOPIA CORRECTING' section includes 'Synergy™ DFW', 'Symfony™ ZXW', 'Symfony™ ZXT', 'Multifocal ZKU', and 'Multifocal ZLU'. Each option is accompanied by a 'Toric II' or 'Toric' label.
- Results:** A table at the bottom right showing 'IOL Details' and 'Residual Astigmatism' with columns for 'IOL Model', 'Orientation', 'Cylinder', and 'Axis'.

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Calculator Page – Data Entry

The screenshot displays the 'TORIC CALCULATOR' interface for TECNIS Eyhance. The page is divided into several sections:

- Surgeon and Patient Information:** Fields for Surgeon Name, Date (3/24/2022), Patient Information, and Patient Age. Eye Selection is set to OD (Right).
- Keratometry:** Fields for Surgically Induced Astigmatism (SIA), Flat K1, Steep K2, and Preop Corneal Astigmatism. Includes checkboxes for 'Include Posterior Corneal Astigmatism' and 'Include Meridian'.
- Biometry:** Fields for Axial Length (mm) and A-constant (119.30). Method is set to 'Optical or immersion'.
- Calculation Preferences:** Fields for SE IOL Power, K Index, and Refractive Cylinder Convention (set to Plus).

On the right side, there is a diagram of a toric lens with meridians labeled: 90°, 45°, 0°, 315°, 270°, 225°, 180°, and 135°. The lens is labeled 'OD' and 'Nasal'.

At the bottom right, the 'Final Results' section contains a table:

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis

Navigation links at the bottom include: PRIVACY POLICY, TERMS OF SERVICE, CONTACT, and LRI CALCULATOR.

¹ Koch DD et al. Contribution of posterior corneal astigmatism to total corneal astigmatism. J Cataract Refract Surg. 2012 Dec; 38 (12): 2080–7

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The screenshot displays the 'TORIC CALCULATOR' interface for TECNIS Eyhance Toric II DIU. The interface is divided into several sections: 'Surgeon and Patient Information', 'Keratometry', 'Biometry', 'Calculation Preferences', and 'Final Results'. A schematic diagram of an eye is shown on the right, with a circular diagram indicating the orientation of the astigmatism (orange line) and the incision location (blue line). The 'Final Results' section is highlighted with a blue border and contains a table of recommended IOL models.

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
<input type="radio"/> DIU300	100 °	+0.38 D	100 °
<input type="radio"/> DIU375	100 °	+0.11 D	10 °
<input type="radio"/> DIU450	100 °	+0.61 D	10 °

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Toric II
DIU

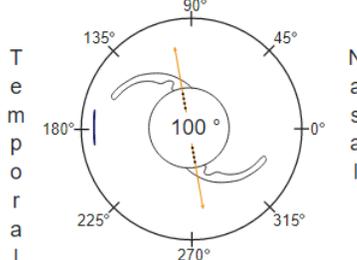
Return to Calculator

Surgeon Name	Patient Information	Date	Patient Age
Dr. Surgeon	XYZ-123-ABC	3/24/2022	65

OD (Right)

T
e
m
p
o
r
a
l

N
a
s
a
l

— (Incision Location) 180°

— Calculated Orientation

IOL Details		Residual Refraction	
IOL Model	Orientation	Cylinder	Axis
DIU375	100 °	+0.11 D	10 °

Calculation Details		
SE IOL Power	K Index	Refractive Cylinder Convention
+20.0	1.3375	Plus

Keratometry and Biometry						
Flat K1 @ Meridian	Steep K2	SIA @ Meridian	AxL Length	Biometry Method	A-constant	PCA
41.25@12°	43.34@102°	0.35@180°	24.11	Optical or Immersion	119.30	No

IOL Details		Residual Astigmatism	
IOL Model	Orientation	Cylinder	Axis
DIU300	100 °	+0.38 D	100 °
DIU375	100 °	+0.11 D	10 °
DIU450	100 °	+0.61 D	10 °

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The screenshot displays the TECNIS Eyhance LRI Calculator interface. A modal dialog box titled "Clear Entries" is centered on the screen, containing the text: "Please confirm that you wish to clear all entries that you have made on this screen. (e.g., OD (Right) & OS (Left))". Below the text are "OK" and "Cancel" buttons. The background interface includes the following sections:

- Surgeon and Patient Information:** Surgeon Name (Dr. Surgeon), Date (3/24/2022), Patient Information (XYZ-123-ABC), Patient Age (65).
- Eye Selection:** Radio buttons for OD (Right) and OS (Left). K Notation: Radio buttons for D and mm.
- Keratometry:** Surgically Induced Astigmatism (SIA) (0.35 D @ Meridian (Incision Location) 180), Flat K1 (41.25 D Flat K1 @ Meridian 12), Steep K2 (43.34 D Steep K2 @ Meridian 102), Preop Corneal Astigmatism (2.09 D). Includes a checkbox for "Include Posterior Corneal Astigmatism".
- Biometry:** Axial Length (24.11 mm), Method (Optical or Immersion), A-constant (119.30).
- Calculation Preferences:** SE IOL Power (20.0 D), K Index (1.3375), Refractive Cylinder Convention (Plus/Minus).
- Final Results:** A table with columns for IOL Model, Orientation, Residual Astigmatism (Cylinder, Axis).

At the bottom of the interface are buttons for "Calculate Results", "Clear Entries", and "Print Results". The footer contains links for "PRIVACY POLICY", "TERMS OF SERVICE", "CONTACT", and "LRI CALCULATOR".

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The screenshot shows the 'Preferences & Support' page of the TECNIS TORIC CALCULATOR. The page is divided into two main sections: 'Preferences' and 'Support'. The 'Preferences' section is further divided into 'Display' and 'Calculation' sub-sections. The 'Display' section includes options for 'Location' (United States), 'Language' (English), 'K Notation' (D), and 'Decimal' (0.00). The 'Calculation' section includes options for 'Refractive Cylinder Convention' (Plus), 'K Index' (-Select-), 'Method' (Optical or Immersion), and 'A-constant' (119.30). There is also a checkbox for 'Include Posterior Corneal Astigmatism'. The 'Support' section lists training and information documents. The footer contains 'PRIVACY POLICY', 'TERMS OF SERVICE', 'CONTACT', and 'LRI CALCULATOR'.

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The screenshot displays the 'TECNIS Family of IOLs' TORIC CALCULATOR interface. The page is titled 'TORIC CALCULATOR United States' and features the Johnson & Johnson VISION logo. The interface is divided into two main sections: 'Calculator' and 'Preferences & Support'. The 'Preferences' section includes options for 'Display Location' (United States), 'Language' (English), 'K Notation' (D), 'Decimal' (0.00), 'Refractive Cylinder Convention' (Plus), 'K Index' (-Select-), 'Method' (Optical or Immersion), and 'A-constant' (119.30). There is also a checkbox for 'Include Posterior Corneal Astigmatism'. The 'Support' section provides links to 'Training and Information Documents', including 'TECNIS® Toric IOL Calculator User Tutorial', 'TECNIS® Toric IOL Calculator FAQ', and 'TECNIS® Toric IOL Calculator Terms of Service'. The footer contains the contact information '(01)05050474708822(8012) - 4.5.2.0 CE' and navigation links for 'PRIVACY POLICY', 'TERMS OF SERVICE', 'CONTACT', and 'LRI CALCULATOR'.

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Johnson & Johnson Surgical Vision, Inc.
31 Technology Drive, Suite 200,
Irvine, CA 92618 USA

AMO Ireland

Block B
Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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AMO Australia Pty Ltd.
1-5 Khartoum Road,
North Ryde, NSW 2113
Australia

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© Johnson & Johnson Surgical Vision, Inc., 2023.

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AMO Switzerland GmbH